

922 Inflation System Connection Instruction

Connecting the Inflation System to Manometer

Remove the red cap which seals the reservoir containing mercury. Attach the free end of reservoir tube securely to air inlet. Save red safety cap for later use. When transporting mercury instrument we recommend replacing the red safety cap.

Please note: It is normal for negligible amounts of residual mercury droplets to accumulate around the air inlet beneath the red safety cap during transportation.

Warning: *Before transporting this instrument, mercury must be stored in the tank and the reservoir lock MUST be switched to the off position to prevent mercury spills.*



55 Commerce Drive • Hauppauge, N.Y. 11788
www.adctoday.com