

Specifications

Size	5"L x 3/4"W x 3/8"H (12.7cm L x 1.905cm W x .9525cm H)
Weight	0.49 oz (14 grams)
Display:	LCD, Liquid Crystal Display
Battery:	1.5v(LR41)
Battery Life:	Estimated 1,500 uses
Measurement Time:	Oral 60 Seconds / Rectal 40 Seconds / Axillary 90 Seconds
Measurement Range:	90.0°F to 109.9°F / 32.0°C to 43.9°C
Accuracy:	0.2°F / 0.1°C between 96.0°F to 107.0°F / 34.0°C to 42.0°C at a room temperature of 64.4°F to 82.4°F / 18°C to 28°C +/-0.5°F/0.3°C between 90.0°F to 96.0°F/32.0°C to 34°C and 107.0°F to 109.9°F/42.0°C to 43.9°C
Measurement Mode:	Real measurement with peak-hold method
Auto Turn Off:	In approximately 8'40" +/- 1 minutes
Memory Feature:	Stores last temperature taken.
Operating Environment:	10 °C to 40 °C, humidity: 15 % to 95 % (noncondensing)
Storage Environment Temp:	-25 °C to +60 °C, humidity: 15 % to 95 % (noncondensing)


WARRANTY: This Thermometer is guaranteed for a period of 24 months from date of purchase. This warranty does not cover batteries, or damage resulting from improper handling or use. Specific legal rights stipulated in your state may guide this warranty. If you have a legitimate claim under this warranty, send item(s) postage paid to Service Center, Attn: Repair Dept., 55 Commerce Dr., Hauppauge, NY 11788. Please include your name and address, phone no., proof of purchase, and a brief note explaining the problem., your return address and \$2.00 to cover shipping and handling.

Visit www.adctoday.com to view this instruction booklet in other languages.



 Onbo Electronic (Shenzhen) Co., Ltd.
No. 497, Ta Laneg Nan Road, Ta Laneg Street,
Baoan District, Shenzhen, China


 Microlife AG, Espenstrasse
139, 9443 Widnau, Switzerland

 Read the instructions carefully
before using this device.

 Type BF applied part

Adtemp™ II 412 Digital Fever Thermometer

USE, CARE & MAINTENANCE

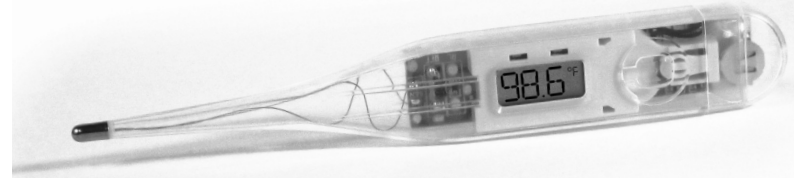

Intended Use:

A digital thermometer is a device used to measure the body temperature of a patient by means of a transducer coupled with an electric signal amplification, conditioning, and display unit.

Safety Information:

- High, prolonged fever requires medical attention especially for young children. Please contact your physician.
- Do not allow children to take their temperatures unattended.
- Read the enclosed instructions carefully to ensure accurate temperature readings. Temperature readings are affected by many factors including exercise, drinking hot or cold beverages prior to taking a reading as well as technique.
- Do not allow children to walk or run during temperature taking. The rigid probe could present a hazard. Oral, underarm, and rectal temperatures vary by approximately 1°F (0.5°C).
- Dispose of batteries in accordance with locally applicable regulations, not with domestic waste.
- **CAUTION:** Do not take an axillary temperature through patient's clothing. Direct probe cover to skin contact is required.
- **CAUTION:** Biting the probe tip while taking a temperature may result in damage to the probe.
- **CAUTION:** Do not autoclave. Please note the cleaning procedures described in this manual
- **WARNING:** This product may contain a chemical known to the state of California to cause cancer, birth defects, or other reproductive harm.
- **WARNING:** Keep battery out of children's reach. Swallowing battery could be harmful. Battery should not be recharged or placed into extreme heat as it may explode.

How to take temperature:

- Place a probe cover carefully over the tip (Figure 1). Select oral, underarm, or rectal as your measurement site based on the individual requiring a temperature measurement.
- Push the Multi-Function button located at the end of the thermometer. The display will read: "188.8 $\frac{^{\circ}\text{F}}{\text{M}}$ ". This is a function check indicating the unit is working properly. The last measured temperature will be displayed for approximately 2-seconds. Next the thermometer will read L°F.(or L°C).
- The °F or °C symbol will flash when the unit is ready to measure temperature. Please note that the "▽" indicates a low battery. Do not expect an accurate temperature reading if this low battery indicator is present. An "ERR" message in the window indicates your unit is not functioning properly.
- Depending on the measurement site, the unit will complete measurement in 40 (rectal), 60 (oral), or approximately 90 (axillary) seconds. When the F° or C° symbol stops flashing and the display reads a consistent temperature, the peak temperature has been reached.
- Read and record the temperature and time for reference. The reading will automatically be stored in the memory.
- Push the Multi-Function button to turn the unit off. This will prolong the battery life. If you do not turn the unit off, it will shut off automatically in approximately 8'40"±-1 minutes.



Figure 1

Oral use: (for children 4 years and older)

- Do not drink hot or cold fluids, exercise or perform other activities that will raise or lower temperature readings when compared to your normal, average temperature. The mouth should remain closed up to 5 minutes before attempting a reading.
- Place the probe tip well under the tongue as indicated (Figure 2).
- Instruct children not to bite down on the probe.
- The peak temperature should be reached in approximately 60 seconds, and a beep will sound. Opening the mouth or improper placement of the probe tip could result in a longer time for a reading.



Figure 2

Rectal use:

- Commonly used for babies, young children or when it is difficult to take an oral or underarm temperature.
- After placing the probe cover on the thermometer, lubricate with a water-soluble jelly or petroleum jelly.

- Gently insert the probe tip no more than 1/2 inch or 1.3 cm into the rectum. Do not force the tip into the rectum if resistance is encountered.
- Dispose of the used probe cover and wash the thermometer as recommended.
- The peak temperature should be reached in approximately 40 seconds, and a beep will sound.

Underarm (Axillary) use:

- Wipe underarm with a dry towel.
- Place probe tip in armpit and bring arm down against the body.
- The probe should be pointing upward in the same direction as your body if possible.
- The time required for an accurate underarm temperature may be 90 seconds or longer depending on placement of the probe and closure of the arm against the body.

Operation of the °F - °C feature (How to change the measuring scale) From the "OFF" position, hold down the Multi-Function button for 5-sec. The thermometer will automatically change from °F - °C or °C - °F. Release the button. You may toggle between °C +°F. After 3-sec. in one scale the °C or °F will flash, indicating ready to measure.

Standards:

Complies with EN 12470-3, ASTM E 1112-00, IEC 60601-1, IEC 60601-1-2
Electromagnetic compatibility: device fulfills the stipulations of standard IEC 60601-1-2
The manufacturer recommends a calibration check every 2 years.

How to care for this product:

- Use of probe covers is recommended to help avoid spreading germs, and to keep the thermometer clean. The thermometer should be cleaned after use even when probe covers are used.
- Clean the thermometer tip and probe only using isopropyl alcohol or soap and water.
- Store the unit in the protective case while not in use.
- Do not store or keep in direct sunlight, or in high temperatures.
- Keep battery away from children.
- Do not open the thermometer.
- Do not drop the thermometer, as this may ruin the unit.
- The manufacturer recommends verifying the accuracy by an authorized laboratory

Parts and Assembly