


Stethoscope ID Tag

Personalized ID Tag

ADC®'s proprietary ID tag fits both Adscope™ and Proscope™ models. Made of two piece injection molded plastic, that snaps tightly together around tubing. To ensure a permanent fit, we suggest adding a drop of glue to each fastening post (See Figure 4).

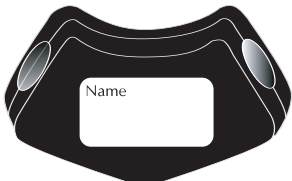

55 Commerce Drive • Hauppauge, N.Y. 11788
www.adctoday.com

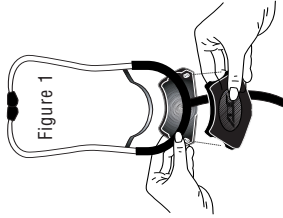


Figure 1

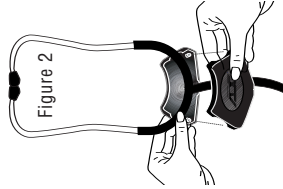


Figure 2

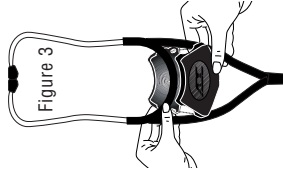


Figure 3



Figure 4

Proscopes™ - Separate the ID tag and center the "Y" tubing in the 3 openings of the rear half, (2 sides and bottom). After positioning as shown, snap front and back halves securely together enclosing tubing completely. (Figure 1)

Adscopes™ - For Adscopes™ that feature Bi-lumen tubing, (Models 600, 601, 602) position ID tag as shown in Figures 3. For all other Adscope™ models, position ID tag as shown in Figure 2. After positioning as shown, snap front and back halves securely together enclosing tubing completely.

Optional - You can permanently lock the tag by adding a drop of glue to each fastener post before snapping closed (Figure 4). Name or initials can be written on rear panel.