




Bird's Eye View Key:
Hardware –

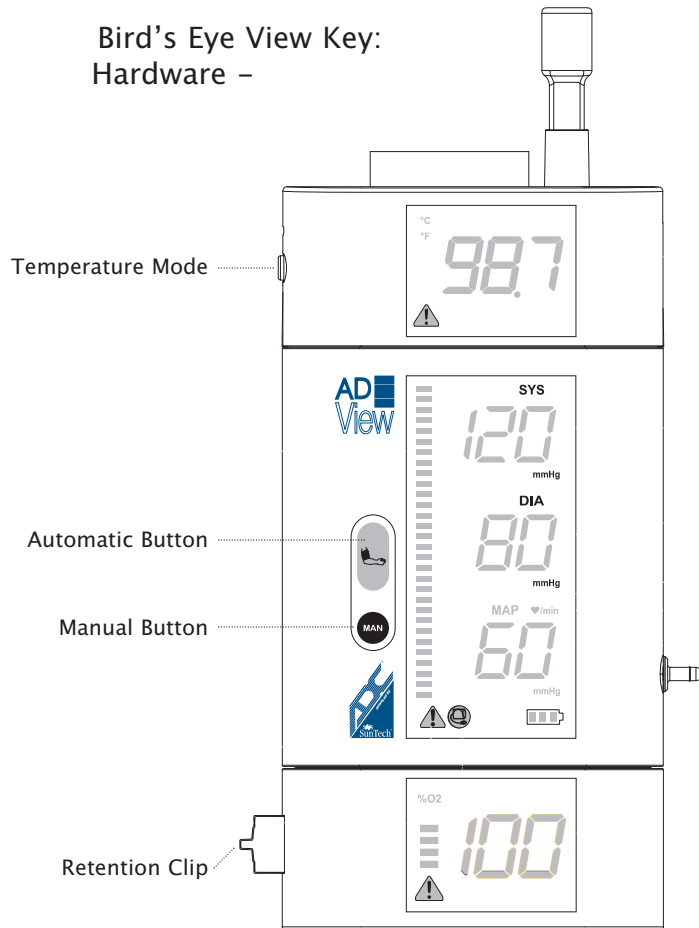


Versions

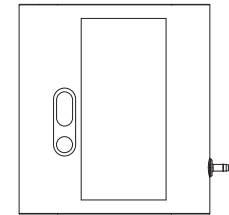


Blood Pressure (BP)

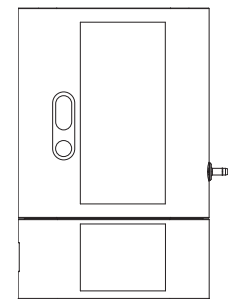

BP and SpO₂

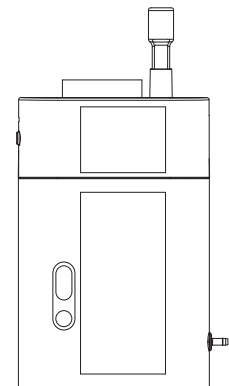


BP and Temperature

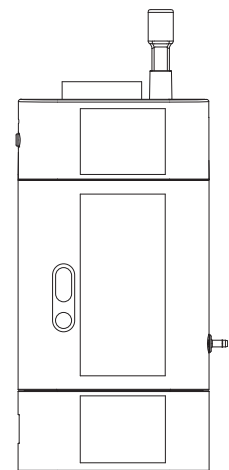


BP, SpO₂, and Temp


Contents & Accessories

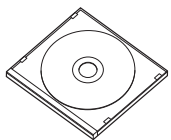


Users Manual CD

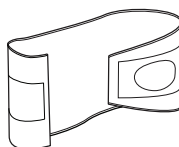

BP Cuff (2)

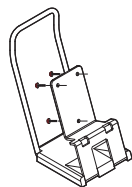


Tabletop Mount

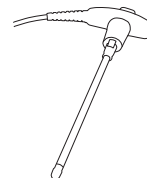

Temperature Probe



Probe Covers

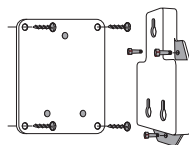


Wall Bracket




Thermometer Module Adapter



SpO₂ Sensor

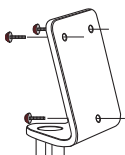

Mobile Stand
(Shipped Separately)

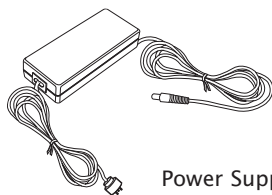

Power Supply




BP Cable

Mounting


The Adviev™ can be mounted in the following ways:

1. Tabletop

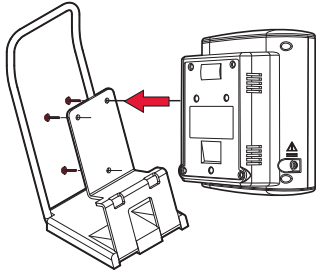


Using the thumbscrews, secure the rear panel of the BP device to the stand.

2. Wall

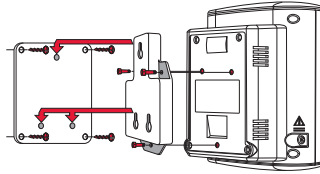

- Attach the bracket to the wall using 4 wall screws, and the basket using 2 screws.
- Attach the wall bracket adapter to the rear panel of the BP device using the 3 screws.
- Position the adapter onto the rivets of the wall bracket and slide the device down until it locks into place.


3. Mobile Stand

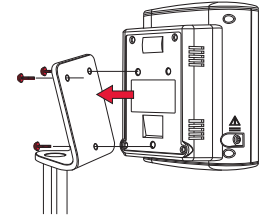

Assemble the mobile stand according to the manufacturer's directions. Using the three thumbscrews, secure the rear panel of the BP device to the stand.


First Time Setup & Operation

1. Assembling the Temperature Module for Use

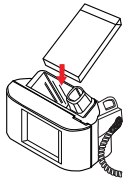


Place the probe cover box into holder.

2. Connecting the SpO₂ Sensor

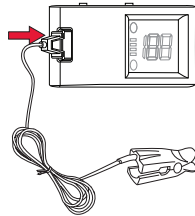


- Flip the retention clip forward, then connect the cable to the device.
- Push the clip backward over the SpO₂ connector to hold the connector in the device.

3. Connecting Power

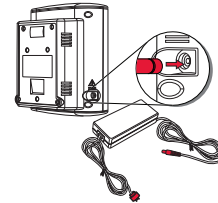

- Connect the power supply to the BP module.
- Connect the power supply to an AC main power source. The device will turn on automatically.

4. Changing the Unit of Temp Measurement

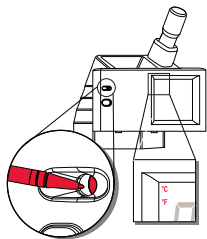


Using the end of a paper clip or similar tool, press the recessed button on the temp module. The icon for the selected unit is displayed.

5. Adjusting Mobile Stand Height


Use the one hand height adjustment on the mobile stand to adjust the height of the device on the pole. Grip the pole immediately below the black stopper.

With your thumb and forefinger, push the stopper upwards and raise or lower the pole height.

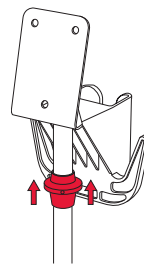

6. Attaching the Quick Reference Card to Basket

Use the four plastic ties to attach the numbered cards to the front side of the basket.

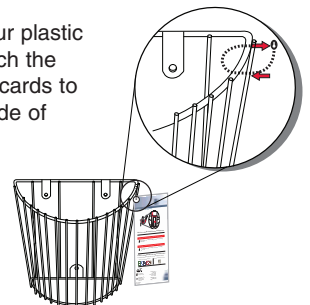

ADC
55 Commerce Drive
Hauppauge, NY 11788



Suntech Medical, Ltd.
Oakfield Industrial Estate
Eynsham, Oxfordshire
OX29 4TS UK

www.adctoday.com/adview
info@adctoday.com

Inspected in the U.S.A.
Made in China
tel: 631-273-9600
toll free: 1-800-232-2670
fax: 631-273-9659

To Register Your Product, visit us at
www.adctoday.com
and follow the links

For questions, comments, or suggestions,
call in the U.S.A. Toll Free:
1-800-ADC-2670