


Assembly Instructions for PROSCOPE™ Teaching Stethoscopes

(Models #671 and #661)

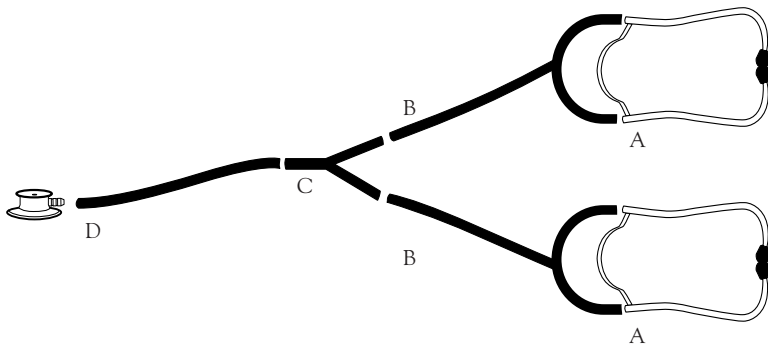

TO ASSEMBLE

- A. Attach “Y” tubing to each set of the binaurals (660-05) as shown in the illustration.
- B. Attach free end of each “Y” tubing to the double pair end of the “Y” connector (#611-20).
- C. Connect one end of the straight tubing (#671-05TBK) to single end of the “Y” connector.
- D. Connect remaining end of the straight tubing to stem of chestpiece (#660-01 or #670-01).

Make sure all tubing connections are firm.

PROSCOPE™ Teaching Stethoscopes
(Models #671 and #661)
Replacement Parts Included

Part Number	Description	Qty.
660-01	PROSCOPE™ Nurse Chestpiece	1*
670-01	PROSCOPE™ Dual Head Chestpiece	1*
611-20	"Y" Connector	1
671-05TBK	PVC Tubing, Straight, 10", Black	1
660-05T	PVC "Y" tubing, 22"	2
660-05	Binaurals	2

***NOTE:** PROSCOPE™ Model #671 includes the 670-01 chestpiece only
PROSCOPE™ Model #661 includes the 660-01 chestpiece only



 ADC
55 Commerce Drive
Hauppauge, NY 11788

tel: 631-273-9600
1-800-232-2670
fax: 631-273-9659

 Attention, see
instructions for use.

 ADC (UK) Ltd.
Unit 6, PO14 1TH
United Kingdom

www.adctoday.com
email: info@adctoday.com

