



Mobile Mount Assembly


1. With base on its side connect upright outer pole to base using the long 6 mm hex screw and washer. Tighten with the large hex key provided. Tighten hardware as securely as possible. (To prevent loosening, periodically check that hardware remains securely tightened.) Reposition base upright.

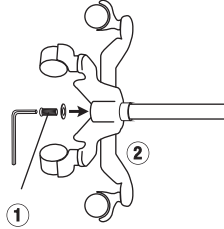


2. Slide stainless steel inner pole inside outer pole with open end up. Make sure shock absorbing spring is inside the outer pole and tighten.

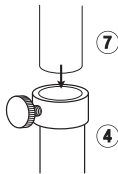

3. Attach metal basket to platform using two 3 mm hex screws. Tighten with small hex key provided.

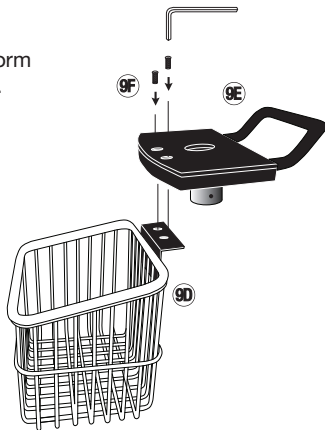

4. Attach the e-sphyg³ to the mounting bracket using short 2 mm hex screws and washers as shown. Tighten with small hex key.

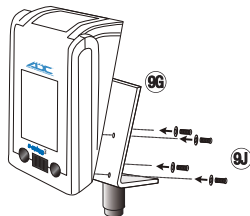

5. Place assembly consisting of the e-sphyg³ and mounting bracket into platform socket, making sure the post goes completely inside collar. Insert short 6 mm hex screw and washer as shown. Tighten with large hex key.




6. Place assembly consisting of the e-sphyg³, cuff storage basket and platform on top of stainless steel inner pole, making sure the pole goes completely inside collar. Use tightening device and small hex key as shown.




Parts List

Mobile Stand Assembly for the 9003M


Ref #	Item #	Description
1	972-101	Long 6 mm Hex Screw and Washer
2	972-080	Base
3A	972-106-1	Single Caster (3)
3B	972-106L-1	Single Locking Caster (2)
4	972-107W	Outer Pole, Winter White
5	972-108	Knob
6	972-109	Spring
6A	972-107A	Gasket
7	972-110	Inner Pole
8	972-111	Tightening Device
9B	752-114	Short 6 mm Hex Screw and Washer
9D	752-119W	Cuff Storage Basket, Winter White
9E	752-117W	Platform, Winter White
9F	752-118W	3 mm Hex Screw (2), Silver
9G	9003M-00	E3 Bracket
9H	752M-117	Small 3 mm Hex Key
9I	752M-116	Large 6 mm Hex Key
9J	9005M-HW-00	Short 2 mm Hex Screws and Washers (4)

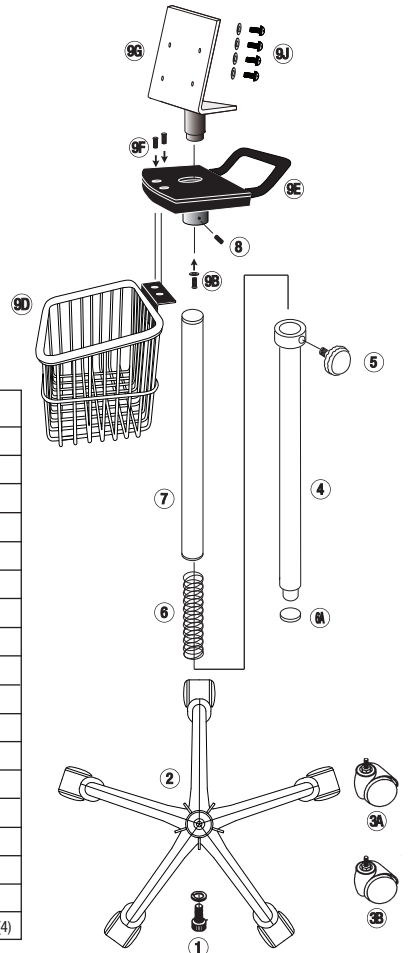

Wall Assembly and Mounting

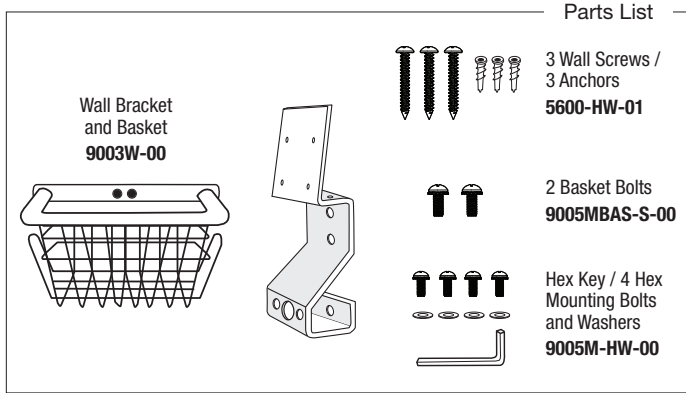


1. Attach e-sphyg³ to wall bracket by aligning the holes on the back of the unit with the four holes on the mounting bracket as shown. Insert the four hex mounting screws with washers through the back of the mounting bracket into the e-sphyg³ and tighten.

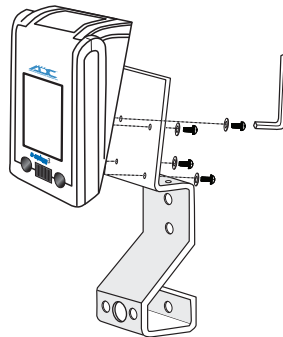

2. Determine the desired location and height on the wall. For optimum viewing, we suggest 60" from the floor to the top of the monitor. Mark the three rear screw positions on the wall making sure bracket is level.


3. Screw the three supplied screws (**5600-HW-01**) firmly into a stud. If stud is not available, install the three supplied anchors first to ensure a secure installation.

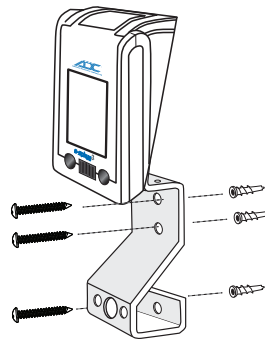

4. Next, attach the basket to the wall bracket. Align the basket screw holes with the bracket screw holes and insert the two supplied basket bolts (**9005MBAS-S-00**) through these openings. Tighten the bolts firmly to secure the basket in place.




Wallboard Assembly and Mounting

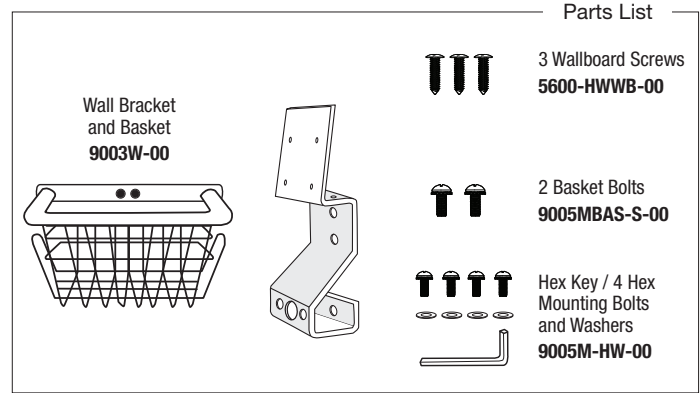


Install the wallboard using the separate Diagnostix Adstation Installation Instructions provided. When you get to "Assemble and Mount Blood Pressure" section, follow the steps below.

1. Attach e-sphyg³ to wall bracket by aligning the holes on the back of the unit with the four holes on the mounting bracket as shown. Insert the four hex mounting screws with washers through the back of the mounting bracket into the e-sphyg³ and tighten.

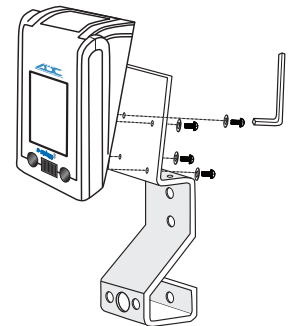

2. Align the wall bracket with the three pre-drilled holes on the wallboard. Screw the three supplied screws (**5600-HWWB-00**) into the wallboard. Avoid over-tightening as this can damage the wallboard.

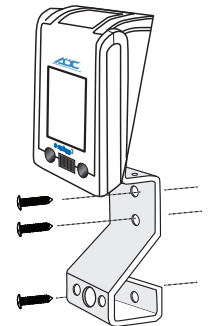

3. Next, attach the basket to the wall bracket. Align the basket screw holes with the bracket screw holes and insert the two supplied basket bolts (**9005MBAS-S-00**) through these openings. Tighten the bolts firmly to secure the basket in place.



Continue to the "Installing EENT Instruments" section on the separate Adstation Installations Instructions.

